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A. Intended Use

A Phlebotomy Bag is used to collect a specific volume of human blood.

Pharmed Medical Industries Company has two models of Phlebotomy Bag without water (Phlebotomy Bag WL) and Phlebotomy Bag with water.

B. Indication

This product is used in treatment of certain diseases in which the hemoglobin rises abnormally or red blood cells number exceeds the upper normal level, such as hemochromatosis and polycythemia.

C. Contraindications

Do not use this product for people who have skin problems, hematoma, or the possibility of blood infection (such as the transfer of bacteria from the surface of the skin to the blood).

D. Cautions and warnings

1. Do not use this product for any other purpose.
2. This product is Single use.
3. The water inside the bag should be clear and transparent.
4. If the package or product is damaged, do not use it.
5. Before use, pay attention to the volume mentioned on the product label.
6. Phlebotomy Bag after blood collection should be disposed in a safe manner.
7. Do not open the carton with sharp things. Sharp tools may damage the product.
8. The Phlebotomy Bag shall not be used more than 10 days after opening the packaging.
9. It is better to use with a physician's advice.
10. Pharmed Medical Industries will not be responsible for bad storage and transportation, abnormal use.
11. The expiry date refers to undamaged and properly stored product.

E. Instruction for use

1. Open the overwrap using the corner flap peeling.
2. After applying tourniquet, the venipuncture area is obvious.
3. Disinfect venipuncture site thoroughly.
4. Hold the Needle hub and twist the cap, the black spot on the needle hub indicates bevel side.
5. After fixing the needle, open venipuncture and allow the bag to fill.
6. For blood concentration cases or increasing process the donor can move patient 's hand gently.
7. For disconnect needle from patient at first clamp the tube then with fixing safety guard, pull tubing smoothly until the needle is locked into the needle guard place sterile gauze over puncture site.
8. Dispose the Phlebotomy Bag in a safe manner.

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*This model of Phlebotomy Bags manufactured by Pharmed Medical Industries Company does not have CE mark.

Note: Gauge 16 Needles, are used in all Phlebotomy Bags.

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I. Manufacture

Name: Pharmed Medical Industries
 Factory: 2nd Bustan, West Hafez Blvd., Eshtehard Industrial Zone, IRAN
 Telefax: +98 (26)3777620-5
 Office: fourth floor, No. 44, Saadat Abad Ave., 1998898638, Tehran, IRAN
 Telefax: +98 (21) 22073102-3
 Email: info@pharmed.co Website: www.pharmed.co

J. European Representative

NephroCan

NephroCan S.r.l., via savoia 78, 00198 ROMA, ITALY
 Tel Office: +39 0685 2371 Fax: +39 0685350187
www.NephroCan.com/EU-Rep