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A. Intended Use

RenaCart (Sodium Bicarbonate Powder Cartridge), RenaBag and RenaBag S (Sodium Bicarbonate Powder Bag) apply for preparation of bicarbonate-based hemodialysis solution. Bicarbonate container must always be used together with a suitable acid concentrate as required by the operator's manual of the corresponding dialysis machine. Also RenaCart, RenaBag & RenaBag S are intended to be used during on-line Hemodiafiltration.

B. Cautions

1. The information in this manual must be read carefully and understood before using the product.
2. **Do not** refill the Cartridge and Bag of Sodium Bicarbonate and **Do not** reuse them.
3. The concentrate in RenaCart, RenaBag and RenaBag S must be used only for a dialysis session.
4. The operation of specially adopted dialysis machine is restricted to persons who are trained on dialysis and who are completely familiar with the operating instructions stated in the operational manual of the corresponding dialysis machine.
5. Do not use if the cap is missed or opened.
6. ISO-quality water meeting the requirements of ISO 23500-3 shall be used to dilute the concentrate to make dialysis fluid

C. Warning

1. **Do Not** use RenaCart, RenaBag and RenaBag S if any damage is found or suspected.
2. **Do Not** use any concentrated hemodialysis solutions other than acid except hemodialysis solutions as required by the operator's manual of the corresponding dialysis machine.
3. **Do Not** initiate dialysis without having followed the instructions the operator's manual of the corresponding dialysis machine regarding any recommended tests to verify the proper composition of the dialysis fluid.
4. **Do Not** perform any unauthorized modifications or alterations to Sodium Bicarbonate Powder Cartridge and Bag or to the dialysis machine as this may results in a malfunction or have the other serious consequences on the safe operation of the dialysis treatment.
5. The products shall be used only by trained nurses (personnel) and/or doctors who are totally familiar with procedure and exactly according to the brochure.
6. **Discard unused solution and residual of powder. Don't re-use the products.**
7. Pharmed Medical Industries will not be responsible for bad storage, bad transportation, abnormal use and use of the product with any incompatible device.
8. The bag and cartridge must be disposed in a safe manner as means as contaminated medical waste and according to any prevailing environmental regulations

D. Description

1. The Bicarbonate cartridge (RenaCart) is made polypropylene (PP) and contain 550g or 650g or 750g Sodium Bicarbonate. The Bicarbonate Bag (RenaBag and RenaBag S) is made Polyamide/Polyethylene (PA/PE) and contain 550g or 650g or 750g Sodium Bicarbonate. This Sodium Bicarbonate powder enables on-line preparation of liquid sodium bicarbonate solution used in a bicarbonate-based dialysis fluid.

2. When the cartridge or bag is placed on the special holder, Purified water conduct by the dialysis machine through the product, producing a saturated solution of sodium bicarbonate. The dialysis machine mixes the sodium bicarbonate solution with Purified water and the acid concentrate to produce a bicarbonate-based dialysis fluid.

3. Both the powder and liquid solution, obtained from Sodium Bicarbonate Cartridges or Bags, meet the demands of the European and US pharmacopoeias.

E. Contraindications and side effects

There are no absolute contra-indications for bicarbonate dialysis. Sodium Bicarbonate Cartridge or Bag must be used only on the directions of a physician who has evaluated all the pertinent features of the product in relation to individual patient.

F. Adverse reactions:

Some undesirable effects related to dialysis can occur, for instance hypotension, nausea, vomiting and cramps.

G. Precautions for use:

1. RenaCart, RenaBag and RenaBag S (Sodium Bicarbonate powder cartridge and bag) which are suitable for all acidic formulae as per the ration stated on the label of container of concentrated Hemodialysis solution.

Attention: RenaCart, RenaBag & RenaBag S contains 550g or 650g or 750g of Sodium Bicarbonate powder which is suitable to dissolve in 8 liters of Purified water and sufficient for one dialysis treatment of 4 to 5h. At a dialysate flow rate of 500ml/min and Sodium bicarbonate concentration of 35 mEq/l.

2. Acid and basic bicarbonate hemodialysis concentrates should only be diluted for short time before use.

3. The composition of the ready-to-use hemodialysis solution must be closely monitored.

4. Use just only for a dialysis session. The Bicarbonate solution is highly susceptible to microbial contamination so discard any unused solution.

5. Powder is non-pyrogenic.

6. Storage: between +5°C and +30°C

7. Protect frost.

H. General instructions for use

See operator's manual of instructions for the corresponding hemodialysis machine.

I. Specific Instructions for installation

RenaCart (Sodium Bicarbonate Powder Cartridge)

1. Switch on the machine as described in corresponding operator's manual.
2. Open the two latches of the cartridge holder
3. Insert the lower cartridge connection into the hole of the lower latch.
4. Press down the upper latch and insert the upper cartridge connection into the hole of the upper latch.
5. Press down the upper latch until you hear a noise indicating that the cartridge is locked in place. Turn the cartridge 90° to ensure the free flow of the liquid.
6. Follow the other instructions in the directions for use of the corresponding haemodialysis machine.

RenaBag and RenaBag S (Sodium Bicarbonate Powder Bag)

1. Remove the cover.
2. Hold vertically with port applied toward the machine.
3. Insert the connector of the bicarbonate bag to its appropriate nozzle of the dialysis machine.
4. Press forward till, click that means complete fixation of the bag to the machine.

Doc. Code	Rev. No.:	Date Of Issue	Page
F72300RD04	R12	99/12/18(08/Mar./2021)	1 of 2

5. The machine will automatically detect the bicarbonate bag and will start dissolving the Sodium Bicarbonate powder to form the bicarbonate solution needed. The bag is sufficient for one hemodialysis session.

J. General instructions to disconnect: After disconnecting the patient, proceed as follows:

RenaCart (Sodium Bicarbonate Powder Cartridge)

1. If the control program of the machine includes the cartridge discharge procedure before the cartridge is removed from the holder after dialysis, perform this procedure first and then remove the cartridge from its holder. Otherwise, proceed as per the following points.

2. Lift the upper latch.

3. Place a finger on the upper cartridge opening, at the same time lift the cartridge out and twist it if appropriate. You can now remove the cartridge.

4. Clean the cartridge holder and close both latches.

5. Proceed further in accordance with the instructions of the directions for use of the corresponding hemodialysis machine.

RenaBag and RenaBag S (Sodium Bicarbonate Powder Bag)

1. After finishing the hemodialysis session, press on the bottom beside the nozzle of bicarbonate bag in the machine for RenaBag, and for RenaBag S remove the cover of seat location of RenaBag S.

2. Emptying the bag from the remaining solution by pressing on the control panel of the software of hemodialysis machine (Empty bag) to remove all the residual solution in the bag before disconnecting the bag from the machine.

3. Remove the Bag.

K: Generic name of the raw material

NaHCO ₃	Sodium Bicarbonate
PE	Polyethylene
PP	Polypropylene
PA/PE	Polyamide/Polyethylene

L. Manufacture

Name: Pharmed Medical Industries

2195

Factory: 2nd Bustan, West Hafez Blvd, Eshtehard Industrial Zone, IRAN

Office: fourth floor, No. 44, Saadat Abad Ave., 1998898638, Tehran, IRAN

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O. Type of hemodialysis machine:

RenaBag shall be used with Fresenius hemodialysis machine 4008 B designed for acid concentrates Hemodialysis solution (Bicarbonate dialysis)

RenaBag S shall be used with Fresenius dialysis machine 5008, 4008 S and designed for acid concentrates Hemodialysis solution.

RenaCart is compatible with the herein Hemodialysis machines designed for acid concentrates Hemodialysis solution:

Dialysis Machine Type	Nipro	Bellco			Hospital		Gambro			B-Braun	
Model	Nipro Sordial 55 Plus	Bellco formula 2000	Bellco formula Therapy	Bellco formula Plus	Hospital Integra	Hospital Montral	AK 95	AK 200	AK 96	B-Braun Dialog	B-Braun Dialog+

P: Product code:

Product	RenaBag 650g	RenaBag S 650g	RenaCart 650g	RenaBag 550g	RenaBag S 550g	RenaCart 550g
Product Code	4315210	4315220	4315310	4315205	4315215	4315305
Product	RenaBag 750g	RenaBag S 750g	RenaCart 750g			
Product Code	4315230	4315240	4315315			

Doc. Code	Rev. No.:	Date Of Issue	Page
F72300RD04	R12	99/12/18(08/Mar./2021)	2 of 2

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Name: NephroCan S.r.l.
Address: NephroCan S.r.l. VIA SAVOIA 78, 00198 ROMA, ITALY