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**A. Intended use**

Intended for the collection of human blood and the preservation of blood compartments.

**A1. Indication**

The phlebotomy bag is used in treatment of certain diseases in which the hemoglobin rises abnormally or red blood cells number exceeds the upper normal level, such as hemochromatosis and polycythemia. The collected blood should be discharged. It cannot be used for any other treatment.

**B. Warnings**

- 1- Inspect overwrap and pack.
- 2- The product is sterile and non-pyrogenic only if the package is intact and all caps are in place.
- 3- Do not use the product unless solution is colorless or faintly yellow, clear liquid, practically free from particles.
- 4- Do not use the product if observe any damage on component.
- 5- Bring the product to a temperature between +15°C and +30°C before use.
- 6- Avoid contact of product with sharp objects.
- 7- Do not vent.
- 8- Risks linked to the reuse: septic risk, coagulation risk, loss of efficacy.
- 9- The PVC complies with the European pharmacopeia. The plasticizer is DEHP the biological products stored and/or circulating in these devices could contain phthalates. (Platelet bag contains TOTM Plasticizer)

**B1. Contraindication :**

.The plasticizer is DEHP and it not recommended for children, pregnant and nursing women

**C. Precautions**

- 1- Pharmed's Blood Bag is single use only. Discard entire set after use.
- 2- If the Primary package is opened more than 10 days, do not use it.
- 3- If the Aluminum package is opened more than 30 days, do not use it.
- 4- Do not remove the needle cap before blood collection.
- 5- Put the filled bags in centrifuge cups with minimum dead space firmly.
- 6- The needle, blood collection tube and any other component should be disposed in a safe manner.

**D. Instruction for use**

- 1- Peel open the transparent overwrap using the corner flap. (Figure 1)
- 2- Place blood bag pack at a distance of ~ 50cm below donor arm, ideally on an automated blood mixer.
- 3- Apply tourniquet or pressure cuff (60 mmHg).
- 4- Identify the appropriate Vein.
- 5- Thoroughly disinfect site of Venipunctures.
- 6- remove the needle cap: Hold the hub and twist the cap. remove the cap in a straight line to avoid any damage to needle tip. The black spot on the needle hub indicates bevel side. (Figure 2)
- 7- Clamp the main tube (Cord) assembled to collection bag.
- 8- Insert the needle into the appropriate vein.
- 9- Do not dangle sampling bag, while is filing. (Figure 3)
- 10- After filling the sample bag, clamp this path and open the clamp on the main tube (cord) assembled to collection bag then break the breakable valve on the donation line with a side to side motion. (Figure 4)

Doc. Code	Rev. No.:	Date Of Issue	Page
F72300RD35	R07	99/12/01 (19/ Feb./2021)	1 of 2

11. During blood collection, occasionally and gently invert or shake blood bag to ensure thorough mixing of blood and anticoagulant.
- 12- Remove the cap of the blood collection tube (tube holder), by pushing the handle of the cap with a thumb to release. Invert the sampling bag to ensure any air is directed to the bottom of the bag. (Figure 5)
- 13- Insert a vacuum test-tube to the Tube holder. Undiluted sample blood is collected. Multiple samples can be taken by exchanging vacuum test-tubes.
- 14- Blood collection done according to capacity on the label.
- 15- After end of blood collection, remove the needle from donor then weld the tube assembled to Uconnector and other component.
- 16- Release tourniquet and withdraw needle from vein and on the donor arm, adhesive will be wound or bandaged on the bleeding site.
- 17- Immediately after bleeding, empty the blood in the main tube (cord) as much as possible by roller into the bag.
- 18- After donation and sampling, move the safety guard to the hub of the needle arrow on safety guard in front of the needle eye. (Figure 6)
- 19- Remove the donation needle, and then gently lower the needle into the barrel of the safety guard until a click is heard.
- 20- Insert the safety guard (needle protector) into the blood collection tube (tube holder) and discard.

#### F. Final phase

Separate segments by sealing appropriate locations of donor tube.

**after needle part seal the tube up to end and be careful not seal on serial number of cord.**

Except Single Blood Bag units, the empty satellite bags may now be used as transfer bags for further processing according to standard compartment processing techniques.

**Note: the using method of membrane port (Blood Injection Port) shown in figure 7.**

#### G.Manufacturer

Name: Pharmed Medical Industries

Factory: 2<sup>nd</sup> Bustan, West Hafez Blvd, Eshtehard, Industrial Zone, IRAN

Telefax: +98(26)377 77620-5

Office: fourth floor, No. 44, Saadat Abad Ave., 19988 98638, Tehran, IRAN

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Name: NephroCan S.r.l.
Address: NephroCan S.r.l. VIA SAVOIA 78, 00198 Roma, Italy .